

Installing web-streamed *Anywhere* viewer

In your browser address bar type the URL: **viewer.gmi.net.au/anywhere**.


Tip...

Save the site to your favourites or onto your favourites toolbar as GMI.

Logging in:

Use your username as your password on first access, at which time you will be prompted to reset it.

Using the *IntelliSpace Anywhere* viewer:

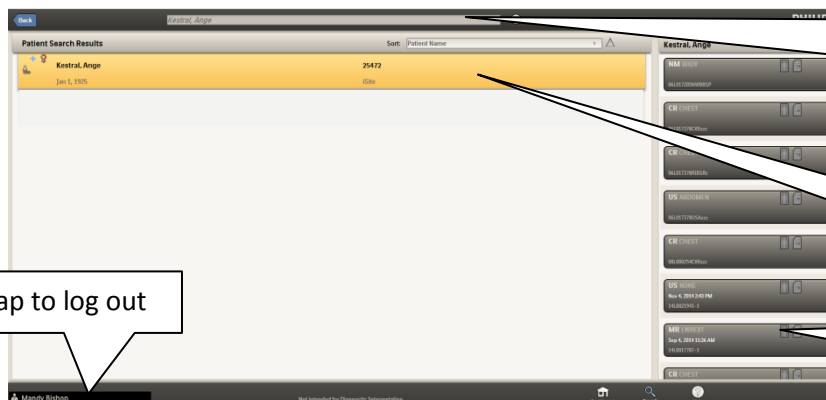

Type the patient surname, first name in the top bar and press enter

Patient options are shown here

Click/tap the relevant study

Click/tap to log out

Image adjustment tools

Report

Imaging history timeline

